


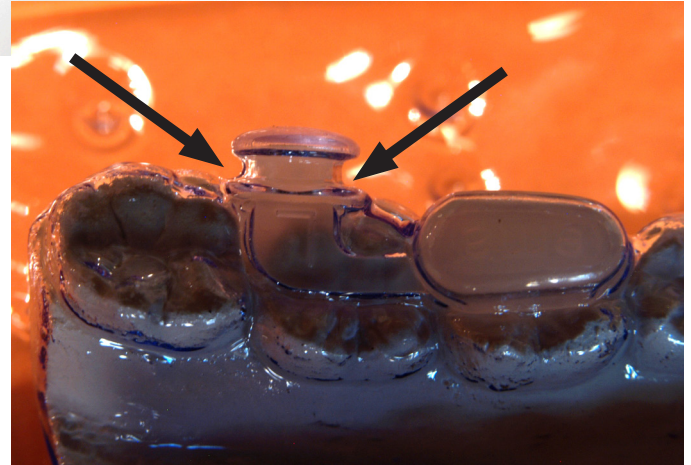


**CORRECT.**

Vacuum forming sheets were heated properly before forming. The material wraps completely around the button.




**OVERHEATED VACUUM FORMING SHEETS.**

Overheated vacuum forming sheets sagged more than the recommended 1/2 inch. Overheating can result in webbing or wrinkling.

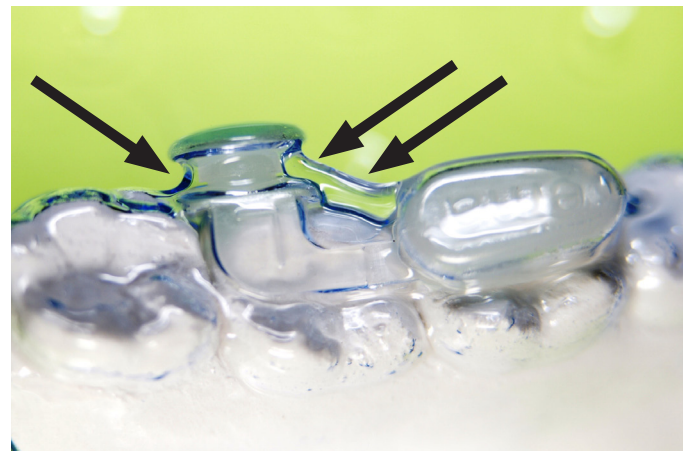

**UNDER HEATED VACUUM FORMING SHEETS.**

Under heated vacuum forming sheet were formed prematurely. Sheets do not wrap completely around the button.

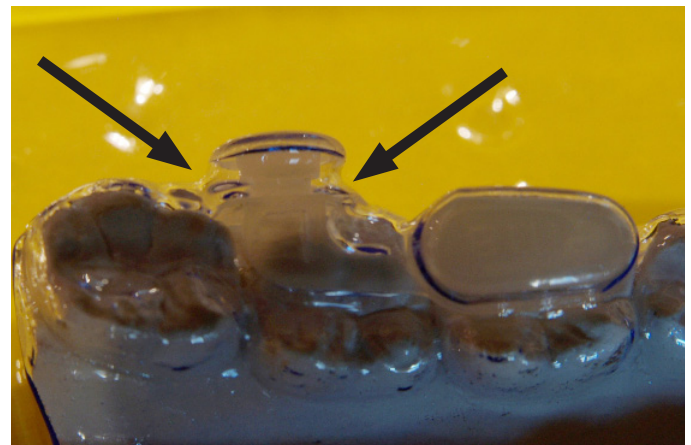

MANUFACTURED BY  
**myerson LLC**  
5106 north ravenswood  
chicago il 60640-2713  
usa  
phone 800 423 2683

EUROPEAN REPRESENTATIVE  
**myerson limited**  
34 railway approach  
station road  
harrow middx, HA3 5AA  
united kingdom  
phone +44 0 20 8863 9044



ema is a trademark of frantz design inc.  
US Patent Numbers; 6,109,265; D611,153 & Patent Pending